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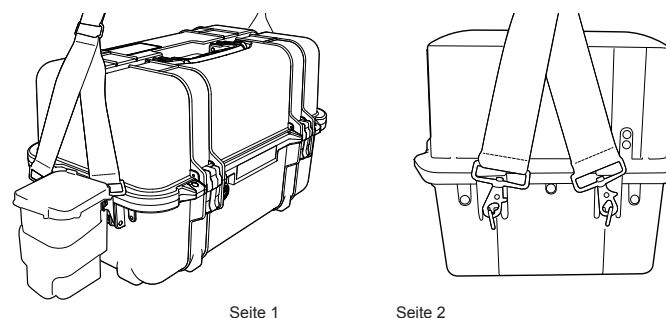
EN


ASSEMBLY INSTRUCTIONS

Please read through the entire instruction sheet and determine which configuration you prefer before beginning any assembly.


To install sharps container bracket:

1. Position the sharps container bracket on either end of the case and slide the tabs of the bracket under the lip of the case as shown in Figure 1A.
2. To attach the container, slide the supplied screws through the holes on the case into the threaded inserts in the bracket as shown in Figure 1B.


HAND TIGHTEN ONLY, DO NOT OVER TIGHTEN.

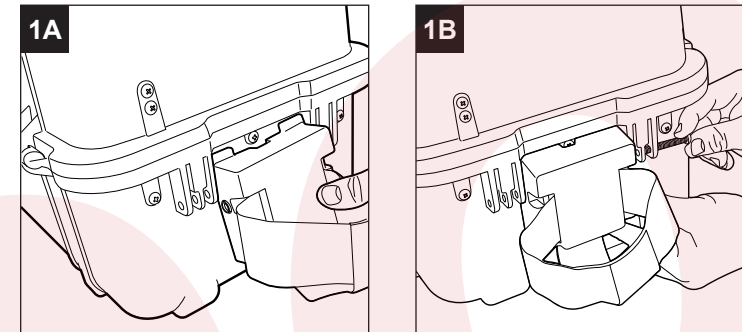**To install handles on one or both ends of the case:**


1. Position the handle so that the tines of the handle fit between the ribs on the end of the case as shown in Figure 2A.
2. To attach the handle, push the supplied pins smooth end first into the holes as shown in Figure 2B.
3. Tap the pin in with a hammer until flush and repeat for the other side.

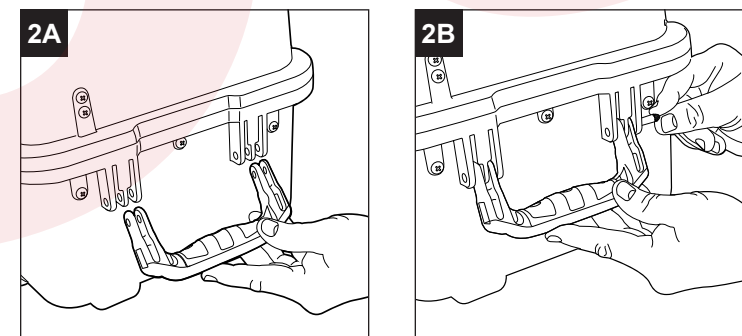**To attach the shoulder strap:**


1. Use a handle pin and one ring as shown in Figure 3A. Repeat on the opposite side and again on the other end of the case until all four rings are installed with four pins.
2. To attach the shoulder strap with the sharps container, follow step 1 except position the ring on the screw between the two outer most ribs before tightening the screw.

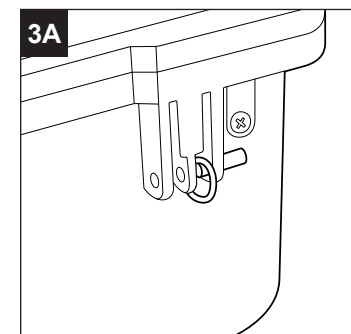**SIDE HANDLE, SHARPS CONTAINER AND SHOULDER STRAP CONFIGURATIONS**


Option 1 - Side handles are installed on both ends. Side handles are used to pull the case from a shelf.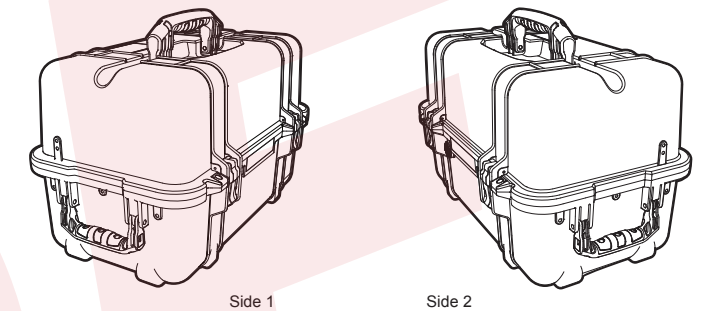


Option 2 - Side handle on one side and sharps container on the other side. Side handle is used to pull the case from a shelf.

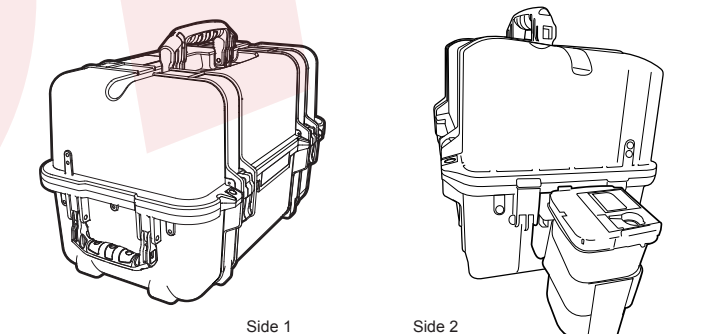Option 3 - Shoulder strap assembly only. Shoulder strap is used to carry the case and pull the case from a shelf.

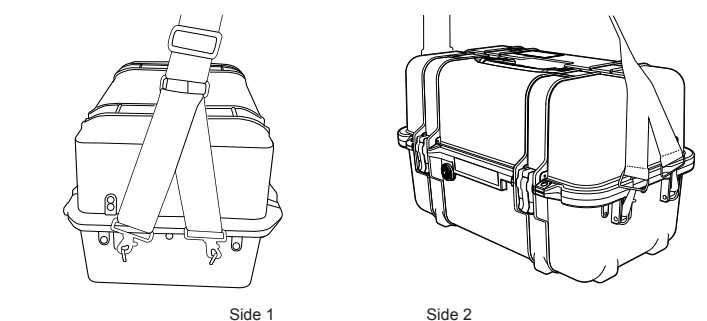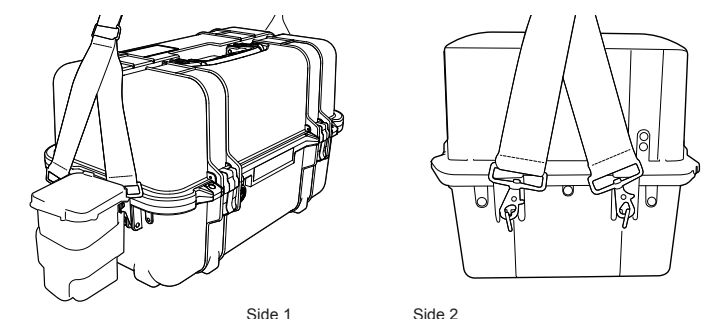Option 4 - Shoulder strap assembly with sharps container on side. Shoulder strap is used to carry the case and pull the case from a shelf.

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